

Blood Collection Reference Guide

Questions? Call 248 301 6917



OR



DON'T SPIN

SPIN

SPIN

Light Blue

Protime/INR

ALL TESTS BELOW ARE ALWAYS A STAT PICKUP

D-Dimer
Antithrombin AG
Factor VIII
Factor XI
Fibrinogen
Lupus Screen
Protein C
Protein S
PTT
Thrombine Time

Note: Tube must be filled to the designated line.

****LAB NEEDS TO BE NOTIFIED UPON COLLECTION****

ROYAL / NAVY BLUE



**ONLY FOR:
LEAD
HEAVY METALS**

Alpha1 Antitryp
ANTI - DNA
ANTI - Thyroid Abs
ASO Titer
CA 125
CA 15-3
CA 19-9
CCP, Ab, IgG
CEA
Ceruloplasmin
CMP
C3, C4
CH50 (frozen)
Cortisol
DHEA-S
Digoxin
Dilantin
Electrolytes
Estradiol
Ferritin
Folate
FSH
Hep C
Beta HCG
Haptoglobin
HSV-I IgG
HSV-II IgG
HIV - I/II

SST

Immunofixation
Homocysteine
IgA, IgE, IgG, IgM
AFP (tumor marker)
ANA Multiplex w/ Reflex
Basic Metabolic Panel
CRP (ultrasensitive)
CMV IgG, CMV IgM
EBV IgG, EBV IgM
ENA Abs (SSA, SSB, SM, RNP)
Free Kappa & Lambda
Valproic Acid (depakote)
Vitamin B-12
Vitamin D 25 Hydroxy
Hepatic Function Panel
Hep A IgG, Hep A IgM
Hep B AB Quant
Hep BS Ag
Hep B Core IgG
Hep B Core IgM
Lyme Disease Ab
Toxo IgG, Toxo IgM

Lipid Panel
LH
Prealbumin
Progesterone
Prolactin
Protein Elect.
PSA
Free PSA
RA
RPR
Syphilis Total AB
Rubella IgG
Sars-Cov 2 IgG
TIBC/UIBC
Transferrin
T Uptake
T3, T4 Free
T3, T4 Total
T7
TSH
Testosterone Free
Testosterone Total

Lavender

CBC w/ Diff
Hgb A1C
Retic Count
Sed Rate

ALL TESTS BELOW REQUIRE A DEDICATED TUBE 1 PER TEST

Hgb Electrophoresis
PTH-Intact
ABO Group & RH Type
Antibody Scr. ID if Pos.
BNP

Note: Tube must be full.

-*AMMONIA -*

Ammonia must be spun, separated, and frozen

Quantiferon TB Kit

Order of Draw:

1. Grey
2. Green
3. Yellow
4. Lavender

Do NOT use butterfly due to limited vacuum in tubes.

Grey

Blood Alcohol
Glucose
Glucose Challenge - 1 Hr
Glucose Tolerance - 2 Hr
Glucose Tolerance - 3 Hr

Notes:
Tube must be minimum half full.
Tubes must have Time of Collection

STAT PICKUP

****LACTIC ACID****

Spin, Separate & Refrigerate
****LAB NEEDS TO BE NOTIFIED UPON COLLECTION****

Red Top

(Clot Activator)

Lithium
Phenobarbital
Tegretol
Theophylline



SPIN

Wait 20 minutes after collection
Centrifuge for 10 minutes at 3500 RPM

URINE CULTURES MUST BE TRANSFERRED INTO A C&S TUBE
WITHIN 20 MINUTES OF COLLECTION