



# 144 Regular condoms

OK to Saint  
Epollod  
9 JUNE 2015

## Manufactured in Malaysia for:

Sax Health Care Pty Ltd  
2 / 6B Railway Avenue  
Oakleigh Victoria 3166 Australia  
Tel: 0414 744705  
www.saxhealthcare.com.au

## Made in Malaysia by:

Karex Industries Sdn. Bhd.  
PTD 7906 & 7907,  
Taman Pontian Jaya,  
Bt. 34 Jalan Johor  
82000 Pontian Johor

CE 0120



Advena Limited, Pure Offices  
Plato Close, Warwick CV34  
6WE UK

LOT



Label Code: SR1441.04



9 328951 004009 >

**144 lubricated (non spermicidal) condoms. 53 mm nominal width.**






**These condoms are made of rubber latex which may cause allergic reaction including anaphylactic shock in some people.**

**These condoms conform to international standard ISO 4074.**

**Do not use condom if packaging is open or appears to be tampered with.**

**Correct use of Sax condoms may help reduce the risk of sexually transmitted infections such as AIDS and also pregnancy. Please read the following instructions carefully.**

## Usage Instructions

- Before using a condom check to see that it is still within the expiry date printed on the foil and the packet.
- Remove the condom from the foil carefully so as not to damage the condom. 
- Check that the condom is the right way up for unrolling.
- The condom should be placed on the erect penis before any contact between the penis and the partner's body. 
- Gently squeeze the tip of the condom to remove the air and make room for semen 
- Roll the condom down the whole length of the penis. Stop to check to see if the condom is too loose or too tight as either of these conditions could cause usage problems. 
- Applying additional water based lubricant, such as Wet Stuff, is recommended before and during sex. Do not use oil-based lubricant such as petroleum jelly or baby oil.
- Discontinue use and consult a doctor if either partner experiences burning or irritation
- Withdraw the penis soon after ejaculation whilst holding the condom firmly in place at the base of the penis, to prevent spillage. Avoid any contact between genital areas until you put on a new condom. 
- Seek medical advice as soon as possible, at least within 72 hours, should a condom leak or burst during use.
- Wrap the used condom in tissue and dispose of it hygienically in a closed rubbish bin. Condoms are non-flushable.
- Condoms are for single use only. Reusing a condom could increase the risk of breakage or infection.
- If you or your partner are using any topical medicines in the genital area please consult a doctor or pharmacist about the compatibility with condoms.
- Store condoms in a cool dry place away from direct sunlight.
- Do not return used condoms to the distributor through the mail