

BEAM

PROFESSIONAL TEETH WHITENING

Thank you for choosing BEAM Professional Teeth Whitening products. All of our whitening solutions are manufactured in a FDA registered facility in the USA. We provide effective and affordable teeth whitening solutions using the highest quality ingredients available.

1. Apply 0.5cc of the clear whitening gel in the top arch of the mouth piece and 0.5cc in the bottom arch spreading the solution evenly with syringe tip. Be careful not to overload mouthpiece.
2. Place mouthpiece in your mouth and bite down gently. Keep tray in mouth for about 15-30 minutes and then rinse out mouth and tray with water. The mouthpiece is manufactured in a "Ready Form" material so there is no need to heat and form.

Use the day after a professional in office teeth whitening treatment to boost results by repeating steps above daily until satisfied and/or use once per month as a maintenance to maximize the longevity of your results.

OPTIONAL - The blue syringe is a remineralization gel and is great for rehydrating the teeth after a teeth whitening treatment. You would follow the same instructions above for the blue syringe except the difference is only keep it in your mouth for 10 minutes then rinse.

DISCLAIMERS:

Keep out of reach of children. Avoid contact with eyes and skin. Consult your dentist before using. Do not use if pregnant or lactating or if you have gum disease or other dental conditions. Discontinue use if pain on teeth or gums occurs. Use only as instructed. Use at your own risk. Purchasing any of our products frees all distributors, manufacturers and affiliates from all liability associated with the use of this system. Do not use if you have allergic reactions to peroxide products. A temporary side effect of tooth sensitivity or mild tissue irritation is possible but usually transient and may last for a day or two. Avoid beverages that contain citrus and hot and cold drinks for the next 24 hours. If you have any adverse reactions terminate treatment and consult your health professional.