

Precept®

Optimizing Safety, Maximizing Comfort

① Wash hands before putting on the mask.

PROPER DONNING OF EARLOOP MASK *as simple as 1, 2, 3*

2



With colored or printed side facing out, and the nose wire at the top, put your fingers through the earloops, position the mask over your nose and mouth. Place the earloops securely around the ears.

3



Pull the mask from the top and bottom to fully open the folds of the mask, and adjust the mask around the face.

4



Gently form (do not pinch) the nose wire over the bridge of the nose, and make any final adjustment.

5



Properly secured earloop mask.